



OMEGA OPTIX

Instruction manual (handling) - for spectacle lenses

The spectacle lenses should be stored in a dry environment at a temperature range of -20 C° to +50 C°

Keep the spectacle lenses in an original storage box.

Never place the lens face down.

For cleaning the spectacle lenses proper cleaning methods and materials should be used - soft wipes, preferably microfiber, never use a coarse wipe.

Never clean the lenses with dry wipes.

Using the solvents containing acetone, methanol, ammonia and other solvents may damage some type of lenses.

Do not wash the anti-slip layer and the lens marker before grinding lens.

Notice:

- 1/ In the case of changing diopters or switching to multifocal lenses, time is needed to get used to it.
- 2/ These spectacle lenses are not intended to protect the user's eyes from injury.
- 3/ Do not use lenses to observe the sun directly.

